

Intersurgical Aquamist™ Humidifier Nebuliser



INTERSURGICAL®
COMPLETE RESPIRATORY SYSTEMS



Intersurgical Aquamist™ Humidifier Nebuliser

The Intersurgical Aquamist™ Humidifier Nebuliser has been designed to deliver accurate concentrations of humidified oxygen quietly. The Humidifier Nebuliser allows the clinician to prescribe a concentration of oxygen which will remain unaffected by the flow rate or the patient's own breathing pattern.

The Intersurgical Aquamist™ Humidifier Nebuliser offers a range of accurate (Ref 2) oxygen percentage concentrations, which may be selected to suit the clinical requirements.

TAMPER RESISTANT OXYGEN CONTROL SETTING

The oxygen concentration selection is made by lifting and turning the top of the nebuliser and aligning the required oxygen percentage with the arrow. This tamper resistant control prevents accidental concentration changes, providing a safe and reliable product.

The Humidifier will operate with a driving gas flow of 5 to 11 l/min dependant on the oxygen concentration required.

QUIET OPERATION

The Intersurgical Humidifier Nebuliser is exceptionally quiet in use - offering a noticeable difference compared to traditional products. This allows for a more peaceful clinical environment.

HIGH HUMIDIFICATION OUTPUT

The combination of an accurate oxygen concentration and a high humidification output provides an ideal solution to humidification requirements.

PRODUCT OPTIONS

The Intersurgical Humidifier Nebuliser is compatible with the most popular standard, sterile water bottles including Baxter, Braun, Fresenius and MPV Truma. It is also available with a 500ml bottle if required.

It can be simply attached to the majority of flow meters via the universal screw thread fitting. The 1500 and 1502 options are supplied with a 1506-001 adaptor enabling them to be fitted to a BOC Flowmeter.

FEATURES

Low operating noise

Accurate delivered oxygen concentrations

High humidification output

Tamper resistant oxygen control setting

Four product options available

BENEFITS

Providing a quieter clinical environment and improved levels of patient comfort (Ref. 1)

Providing safe and reliable oxygen delivery

Optimum level of humidification

Preventing accidental oxygen percentage change

Suitable for hospital and homecare use

Code: 1500 Intersurgical Aquamist™ with adaptor



Box Qty: 24

Code: 1502 Intersurgical Aquamist™ with bottle and adaptor



Box Qty: 25

Code: 1508 Intersurgical Aquamist™



Box Qty: 24

Code: 1509 Intersurgical Aquamist™ with bottle



Box Qty: 25

Intersurgical Offer:

- A Wide Product Range
- High Quality Products
- All Products CE Marked
- Fast, Efficient Service
- Reliability
- Excellent Value

Deutschland

T 02241 311063 F 02241 313143
info@intersurgical.de

France

T 01 48 76 72 30 F 01 48 76 91 38
intersurgical@intersurgical.fr

España

T 91 653 96 11 F 91 653 99 23
mail@intersurgical-es.com

Portugal

T 01 498 98 80 F 01 498 98 89
mop29707@mail.telepac.pt

Lithuania

T 37017 54541 F 37098 52773
enquiries@intersurgical.lt

Nederland

T 0413 243860 F 0413 243869
info@intersurgical.demon.nl

United Kingdom

Crane House, Molly Millars Lane
Wokingham RG41 2RZ

T + 44 (0)118 9656 300
F + 44 (0)118 9656 356
enquiries@intersurgical.co.uk
www.intersurgical.com